

# WASHING & CARE INSTRUCTION FOR CLOTH REUSABLE FACE MASK

<p>You have been given two masks.</p>	<p>One to wear, and one to wash.</p>	<p>Swap masks each day so that you always have a clean mask to wear.</p>

## TO CLEAN THE MASK

<p>Scrub the mask with soap and hot water before disinfecting.</p>		

## TO DISINFECT THE MASK

<p>Place in a pan with a small amount of clean water. Just enough water to completely cover the masks.</p>	<p>Bring the water to the boil</p>	
<p>Allow to cool</p>	<p>Remove mask from the pan</p>	<p>Hang the mask to dry in the sun</p>